

iSED® PRO Test Card

Exclusively for the iSED Pro Erythrocyte Sedimentation Rate Analyzer



REF 120-04000, 120-07500, 120-12500, 120-25000, 120-50000



INTENDED USE

The iSED PRO Test Cards comply with ISO 7816 standards and are intended for use exclusively with the iSED PRO Automated ESR Analyzer. The Test Cards contain test credits which are required to process samples on the iSED PRO.

MATERIAL

Polyvinyl Chloride (PVC)

STORAGE & STABILITY

Storage temperature when device is not in use -35°C to $+80^{\circ}\text{C}$

Operation temperature $+10^{\circ}\text{C}$ to 30°C

PROCEDURE

For a detailed procedure, see the iSED PRO Operator's Manual (120-09-007).

To download test credits from the Test Card:

1. Insert the Test Card, vertically with the printed side to the left, into the smart card reader.
2. Once inserted, the credits will automatically download onto the instrument and the analyzer will indicate that the download is complete via a message on the screen.
3. Total credits available will include the newly downloaded credits and any residual credits prior to download.
4. Once all credits have been downloaded onto the instrument, the Test Card can be removed and discarded.

Note: If the instrument has negative test credits and additional test credits have been downloaded from the Test Card, the total number of available test credits will be reduced by the total negative credits.

Warning: Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the EU Member State or the Regulatory Authority in which the user and/or patient is established.



ALCOR Scientific LLC
20 Thurber Boulevard
Smithfield, Rhode Island 02917, USA
(Phone) 800.495.5270 □ 401.737.3774
WWW.ALCORSCIENTIFIC.COM



Scan for languages



EMERGO EUROPE
Westervoortsedijk 60
6827 AT Arnhem
The Netherlands



MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland