



# iSED® PRO TEST CREDIT TRANSFER PROCEDURE

## PURPOSE:

Transfer cards will be issued by ALCOR® Scientific Technical Support at their discretion to allow customers to continue using their purchased test credits without interruption by instrument service.

## SCOPE:

This document describes the procedure to be followed for transferring unused test credits from an iSED PRO analyzer requiring service to a loaner/replacement iSED PRO analyzer.

## MATERIALS:

1. Test Credit Transfer Card (issued by ALCOR Scientific upon request)

## PROCEDURE:

1. Both iSED PRO analyzers must be powered on.
2. Insert Transfer Card into Test Card Reader slot of the customer's iSED PRO, with card information facing left. The Card Reader is located inside the bottle compartment, to the left of the bottles. The iSED PRO analyzer will display a screen confirming the action to take.  
**NOTE: If the Transfer Card has credits remaining, confirming the transfer will overwrite the credits on the iSED PRO.**
3. Press "Confirm" to transfer the credits from iSED PRO to the card.
4. Remove transfer card from Test Card Reader when prompted on the screen. The iSED PRO analyzer should indicate "0" credits.
5. Insert Transfer Card into Test Card Reader slot of the loaned/replaced iSED PRO, with card information facing up. The iSED PRO User Interface will display a screen confirming the action to take.
6. Press "Confirm" to transfer the credits from the card to iSED PRO.
7. Remove Transfer Card from Test Card Reader.
8. The loaner/replacement iSED PRO analyzer is now ready to test of samples.

