

# iSED® Test Card

Exclusively for the iSED Family of Erythrocyte Sedimentation Rate Analyzers



REF

112-00250, 112-00500, 112-01000, 112-02000  
112-03000, 112-05000, 112-10000, 112-20000

CE

IVD

## INTENDED USE

The iSED Test Cards comply to ISO 7816 standards and are intended for use exclusively with the iSED Family of Analyzers. The Test Cards contain test credits which are required to process samples on these analyzers: iSED, miniSED®, iSED ELITE, and MIZAR®.

## MATERIAL

Polyvinyl Chloride (PVC)

## STORAGE & STABILITY

Storage temperature when device is not in use -35°C to +80°C

Operation temperature +10°C to 30°C

## PROCEDURE

For a detailed procedure, see the iSED Operator's Manual (112-09-043), iSED/iSED ELITE Operator's Manual (222-09-007), miniSED Operator's Manual (1017-09-001), or MIZAR Operator's Manual (115-09-011).

### To download test credits from the Test Card:

1. Insert the Test Card, with the arrow facing upward and forward, into the smart card reader.
2. Once inserted, the credits will automatically download onto the instrument and the analyzer will indicate that the download is complete via a message on the screen.
3. Total credits available will include the newly downloaded credits and any residual credits prior to download.
4. Once all credits have been downloaded onto the instrument, the Test Card can be removed and discarded.

**Note:** If the instrument has negative test credits and additional test credits have been downloaded from the Test Card, the total number of available test credits will be reduced by the total negative credits.



ALCOR Scientific LLC  
20 Thurber Boulevard  
Smithfield, Rhode Island 02917, USA  
(Phone) 800.495.5270 □ 401.737.3774  
[WWW.ALCORSCIENTIFIC.COM](http://WWW.ALCORSCIENTIFIC.COM)



Scan for languages



EMERGO EUROPE  
Westervoortsedijk 60  
6827 AT Arnhem  
The Netherlands



MedEnvoy Switzerland  
Gotthardstrasse 28  
6302 Zug  
Switzerland